EN

CLEANING AND MAINTENANCE INSTRUCTIONS FOR SURGICAL INSTRUMENTS

This instructions for use covers surgical devices and accessories, tools & instruments made of Stainless steel or Titanium alloy (hereby under: "Instruments")

Attention: For your own safety, please wear personal protective equipment (gloves, glasses, mask).

Pre-Cleaning

- Disassemble the instruments if required.
- Soak all instruments immediately after use in a detergent and disinfecting solution, preferably an enzymatic cleaning solution, with a pH level between 6-9, prepared with lukewarm water for 5 minutes.
- Scrub the instruments under running water with soft nylon brush to remove any remaining blood or debris.
- Rinse under tap water (at least 1 min).
- Place the instruments in a kit, support or rack to avoid any contact between them during the next cleaning procedure.

Cleaning Procedure

Manual Cleaning

- Prepare an ultrasonic bath with a cleaning solution at a concentration and temperature specified in the detergent manufacturer's instructions.
- Immerse the instruments completely and activate the bath for at least the recommended time in the detergent manufacturer's instructions.
- Rinse under tap water (at least 1 min).

Alternative: Automated Cleaning

- Place the rack in a washer-disinfector and apply a cleaning procedure according to the manufacturer's recommendations.

Drying and Sterilization

- Dry on a single-use non-woven cloth or through a drying cycle of washer-disinfector or with filtered compressed air.
- Inspect the instruments and discard those with defects. Repeat cleaning if required.
- Assemble the instruments if required.
- Place the instruments in a kit, and pack in sterilization pouches.
- Steam sterilize according to the table below. Do not exceed the recommended temperature specified.
- Keep inside the sterilization packaging in a dry and clean environment.

Cycle type	Temperature	Exposure time (min.)	Drying time (min.)
Gravity displacement	135°C / 275°F	10 minutes	30 minutes
Pre-vacuum	132°C / 270°F	4 minutes	20 minutes

Recommendations

- Cutting tools should be used for a maximum of 30 uses.
- Distilled water should be used in order to prevent water stains.
- For all metal instruments, it is recommended to use antiseptic disinfecting and cleaning agents. They should be free of: Aldehyde, Ethanolamine, chlorine and acid.
- Use only autoclaves that meet the requirements of EN 13060, EN 285.
- Use a validated sterilization procedure according to ISO 17665.
- For Automated cleaning procedures, use a washer-disinfector approved according to EN ISO 15883.

Key To Codes Used

	Batch code
	Catalog number
	Manufacturer
	Date of manufacture
	Caution, consult accompanying documents
	Caution: U.S. federal law restricts this device for sale by or on the order of a dental professional.

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